GENbox

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GENbox

Atmosphere generators and incubation jars

PRESENTATION

REF 96 124	GENbox anaer 10 aluminium sachets each containing 1 generator + 1 package insert
REF 96 125	GENbox microaer 10 aluminium sachets each containing 1 generator + 1 package insert
REF 96 126	GENbox CO ₂ 10 aluminium sachets each containing 1 generator + 1 package insert
REF 96 127	GENbox Jar 2.5 l
REF 96 128	GENbox Jar 7.0 l

ADDITIONAL MATERIAL

- Anaer indicator 50 strips (Ref. 96 118)

STORAGE CONDITIONS

Store 2-25°C in a dry place.

PRINCIPLE

GENbox generators function without the addition of water or a catalyst (no hydrogen is produced). They provide the user with a high level of practicability and security.

GENbox anaer, microaer and CO₂ generator sachets all contain the same chemical compounds i.e. activated charcoal, sodium ascorbate and other organic and inorganic compounds. The gas compositions obtained in the jar (oxygen and carbon dioxide) are adjusted by the quantity of chemical compounds contained in each sachet, which absorb oxygen and release carbon dioxide.

THEORETICAL CONCENTRATIONS:

	Oxygen concentration	Carbon dioxide concentration
GENbox anaer	< 0.1% after 2.5 hours	> 15% after 24 hours
GENbox microaer	From 6.2 to 13.2% after 1 hour	From 2.5 to 9.5% after 24 hours
GENbox CO ₂	Not available	From 3.5 to 9.0 % after 24 hours

USE

- Setting up:

Petri dishes:

Place the inoculated plates in the jar.

GENbox Jar 2.5 l uses one generator and can incubate a maximum of 12 x 90 mm plates.

GENbox Jar 7.0 l uses three generators and can incubate a maximum of 42 x 90 mm plates.

Note: It is recommended to use vented plates.

Strips:

Place identification strips, for example API strips, in the jars after previously adding water into the individual API incubation trays.

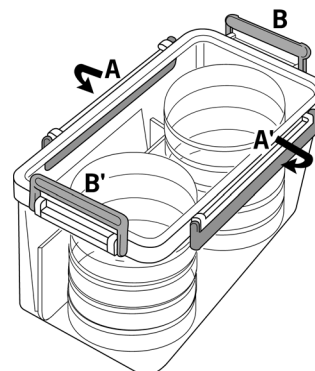
- With the GENbox anaer, also place in the jar an anaerobic atmosphere indicator. Check that it is visible and that it does not touch the inner side of the jar.

The anaerobic atmosphere indicator controls that the anaerobic reaction has been correctly performed and has been maintained throughout incubation.

- Open the aluminium sachet without using scissors or other cutting devices. Take out the required number of generators and place them in the jar.

WARNING: the reaction starts as soon as the generator comes into contact with the air. The time between opening the aluminium sachet containing the generator and closing the jar must be as short as possible. Prolonged contact with air leads to loss of activity resulting in insufficient atmosphere being generated in the jar.

- To close the jar :
 - Align the lid properly onto the jar.
 - Simultaneously** press the two opposite handles down. First A and A' together followed by B and B' together.
 - The jar is now hermetically sealed.

- To open the jar (after incubation) :

-Simultaneously unclip the two opposite handles (direction upwards). First A and A' together followed by B and B' together.

-Take one corner of the lid (right angle) and gently pull up to release vacuum and gases. Then remove the lid completely.

- After use, cool GENbox anaer generator to room temperature after removing it from the jar. CAUTION:

After opening the jar and examining the cultures, if a second incubation is necessary, then new generators will be required.

WARNINGS AND PRECAUTIONS

- Do not use reagents after the expiration date.
- GENbox jars can only be used with GENbox generators.
- Never open the aluminium sachets containing the generators before use.
- When handling generator paper sachets, wear gloves or wash hands thoroughly with soap and water immediately after use.

- If the ingredients in the generator come into contact with the eyes, rinse thoroughly with water for at least 15 minutes.
- Never add water to the generator paper sachet as the reaction will be inactivated.
- Never incubate jars at a temperature of more than 45°C.
- With GENbox anaer, the reaction is exothermic. The temperature of the generator can therefore reach 65°C.
- GENbox jars can be cleaned and disinfected according to the following procedure:
 - by immersion in a 60% alcohol bath for 5 to 10 minutes at 15-25°C
 - by immersion in water for 30 minutes at the following recommended temperatures:

2.5 l Jar – box and lid	80 to 85°C
7.0 l Jar - box	80 to 85°C
7.0 l Jar - lid	60°C
Silicon seal	100°C

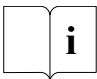
WASTE DISPOSAL

All the generators used must be discarded with other laboratory waste.

LITERATURE REFERENCES

1. ARAKI H, MISAWA S., OGURI T. et al. -Evaluation of Anaeropack system, a new anaerobic culture method- 1995, World Congress on Anaerobic Bacteria and Infections, Puerto Rico.
2. DELANEY M.L., ONDERDONK A.B. - Evaluation of the Anaeropack system for growth of clinically significant anaerobes - 1997, *J. Clin. Microbiol.*, vol.35, n°3, p. 558-562.
3. VAN HORN K., WARREN K., BACCAGLINI E. - Evaluation of a new anaerobic atmosphere generation system - 1996, 96th General Meeting of the ASM, New Orleans.
4. VAN HORN K.B - Evaluation of a new microaerophilic atmosphere generation system for growth of *Helicobacter pylori* - 1997, 97th General Meeting of the ASM, Miami.

INDEX OF SYMBOLS

Symbol	Meaning
REF	Catalogue number
	Temperature limitation
	Use by
LOT	Batch code
	Consult instructions for use
	Keep dry
	Contains sufficient for <n> tests.

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Printed in France

The logo is a registered and protected trademark of bioMérieux sa or one of its subsidiaries.

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